

SAFETY LANCETS
BLOOD SAMPLING

Easy to handle: Intuitive 2 step operation

Sterile: Needles are gammasterilized, fully covered and protected with a sterility tab

Efficient: an adequate blood sample with just a single push of the button



Safe: No possibility to reuse

Gentle: Electro-polished and silicone-coated needle is linearly driven to assure lower pain and quick healing

Reliable: Risk of misuse or receiving an incorrect blood sample is eliminated through a unique, patented, selfdestructive activation mechanism




KEY FEATURES OF ACTI-LANCE

- **Easy** to handle: intuitive activation, low firing force, no lancet loading required, 2 step operation only
- Minimized pain: ultra sharp and silicon coated needle, double spring mechanism to ensure high puncture speed and eliminate needle vibrations
- Sufficient to lance once: different color-coded versions, right needle exposure length, adequate blood flow
- Self-destructive mechanism excluding possibility of reuse, sterilized reliable sterility protector



INSTRUCTIONS OF USE

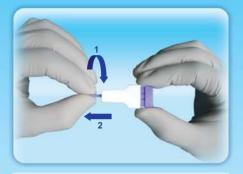

Twist off the protective cap and pull it straight out.



Place Acti-Lance on the desired puncture site and press the top push button to activate the device



Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

TECHNICAL DATA & ORDERING INFORMATION

Acti-Lance Safety Lancets are available in three different color-coded versions to meet specific blood sample size applications and address a variety of skin types



Collecting blood has never been so painless! Acti-Lance Lite is ideal for blood glucose monitoring purposes and is especially recommended for all patients with delicate skin including children.





1.8 mmPenetration depth

23GNeedle

Acti-Lance Universal is perfect for most of the common tests done with capillary blood including blood glucose, cholesterol, hemoglobin, blood group classification, coagulation tests, virology tests and many others.



Acti-Lance Special assures larger blood samples for the widest variety of clinical purposes. It is also perfect for heel sticks on neonates as well as finger sticks on adults in case where the test requires more blood.

> for more information go to: www.acti-lance.com

