

HAE[®]MEDIC
Plus

Paediatric



**Gentle and safe blood-sampling device
for heelsticks in premature infants,
newborns and babies.**

HaeMedic

Features


- Single use safety lancet
- Permanently retractable blade
- Cannot be reloaded or reused
- Quick spring action
- Controlled blade depth - preventing deep punctures
- Electro polished needle tip

Benefits

- Helps prevent accidental needle sticks and eliminates cross - contamination
- Extra safety
- Increased comfort
- Easy to use



1. Select the site and cleanse thoroughly.

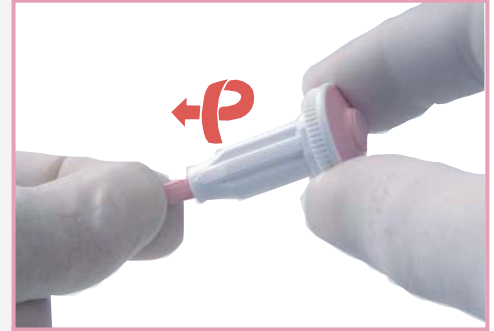

2. To remove the protective cover, turn and twist in one movement.



3. Hold between the fingers and place against the site without touching the button. Then fire by pressing the button.



4. After use, discard in appropriate sharps container or according to local disposal ordinance.



Description	Depth	Width	Packaging
Haemolance Plus Paediatric	1,20 mm	1,50 mm	150/Box

HaeMedic
 Företagaregatan 18,
 266 32 Munka Ljungby
 Sweden
 Phone: +46 431-43 20 30
 Fax: +46 431-43 00 30

HaeMedic
 P. O. Box 806
 Wayzata, MN 55391-0806
 USA
 Phone: 763 404 9651
 Fax: 763 404 5997

E-mail: info@haemedic.se www.haemedic.com