




## Instructions of use

### 1. Twist

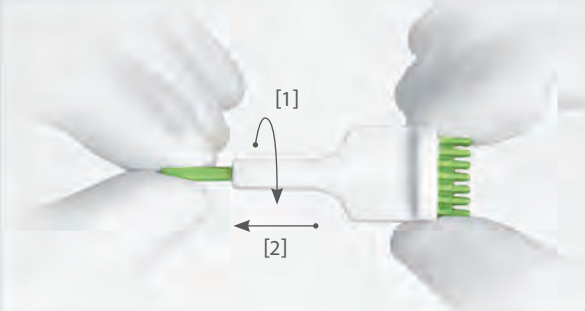

- Twist off the protective cap and pull it straight out.
- Die Sicherheitskappe abdrehen und gerade herausziehen.
- Gire la tapa protectora de la lanceta y retírela.
- Rode a cápsula de protecção e de seguida retire-a.
- Dévissez le capuchon et retirez le parallèlement à l'axe du piqueur.
- Снять защитный колпачок.
- قم بلوي الغطاء الواقعي ثم اسحبه للامام.

### 2. Press



- Press the MediSafe Solo body firmly against the puncture site and press the push-button to activate the device.
- Die MediSafe Solo Lanzette fest gegen die entsprechende Einstichstelle ansetzen und den Auslöseknopf betätigen um sie zu aktivieren.
- Coloque la lanceta MediSafe Solo en la zona de prueba y presione el botón a fin de activar la lanceta.
- Posicione a lanceta MediSafe Solo de maneira firme no local da picada e pressione o botão para activar o dispositivo.
- Appuyez le MediSafe Solo contre la zone de ponction et pressez un bouton pour actionner le dispositif.
- Приложите прибор к месту укола и резко нажмите большим пальцем на кнопку.
- قم بعملية الوخز بالضغط على زر الغطاء.

### 3. Collect



- Gently apply intermittent pressure near the puncture site to obtain the required blood volume.
- Um die gewünschte Blutmenge zu erhalten, mit kurzen Unterbrechungen sanft neben die Einstichstelle drücken.
- Apriete de manera suave y discontinua la zona circundante al área de inyección para obtener el volumen adecuado de sangre.
- Aplique cuidadosamente, uma pressão intermitente junto da zona de picada para obter o volume de sangue necessário.
- Effectuez une pression intermittente près de l'endroit piqué pour obtenir le volume de sang nécessaire.
- Мягко нажимая на место укола, возьмите необходимое количество крови.
- اضغط برفق بجانب مكان الوخز للحصول على كمية الدم المطلوبة.

## MediSafe Solo

29G  
Needle Tip

Penetration Depth 1.5 mm  
Cat. No. 7130  
Shelf Box 200ct



Comfort  
MAX



Recommended use:  
Perfect for blood glucose monitoring.

Sterile, single-use, safety lancets

Sterile Einmalstechhilfen

Lancetas estériles desechables

Lancetas de segurança estereis de uso único

Lancettes jetables, stériles, garantissant la sécurité

Одноразовый безопасный ланцет, стерильный

أداة وخز معقمه ذات الأستخدام الواحد

## MediSafe Solo

23G  
Needle

Penetration Depth 2.0 mm  
Cat. No. 7140  
Shelf Box 200ct



Recommended use:  
Universal application – for testing glucose, hemoglobin, cholesterol, blood type, coagulation, blood gas and many others.



## Product features

CE 0344

**Easy to use.** Concaved top button for thumb activation. Wide & long safety cap for easy removal. V-shape design for convenient finger recess. 2-step operation. Intuitive use. No lancet reloading required. Minimum training for new users.

**Efficient.** Precise design and perfect geometry of the needle assures optimal blood sample collection at the minimum pain level.



# Comfort MAX

**Sterile.** Needles are sterilized with Gamma irradiation and protected with a safety tab.

**Safe.** Single use. Additional self-activation prevention system. Multiple sticks eliminated.

**Patient-friendly.** Ultra sharp, thin needle. Electropolished and silicone-coated. Maximum speed of penetration. Low puncture force.



# BOX 200ct



## Safety lancets



Manufactured by:  
HTL Lancet Sp. z o.o.  
Lotnicz 21h,  
99-100 Łęczycza, Poland  
T: +48 22 587 05 72, F: +48 22 811 05 36  
info@htlancet.pl • www.htlancet.pl



Distributed in the US by:  
HTL-STREFA, Inc.  
3005 Chastain Meadows Pkwy, Suite 300,  
Marietta, GA 30066, USA  
T: +1 770 528 0410, F: +1 770 528 0411  
info@htl-strefa.com • www.htl-strefa.com

520/MS/01/2011